


NC Hoof Crack Repair Kit

Use anytime a quarter or toe crack needs to be stabilized.

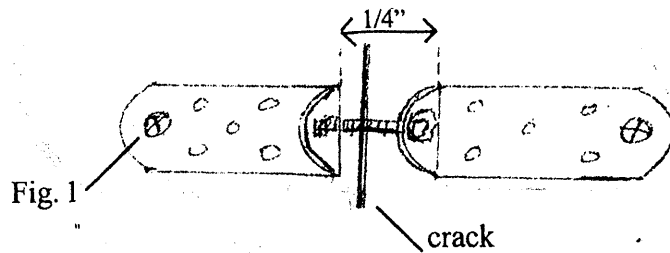

The object of the patch is to stop the crack from moving when the animal bears weight on the affected foot, allowing the animal to be used as the crack grows out.

The patch can be safely applied 3/4" below the hair line.

Start the #4 screw holes with an ice pick or drill probe. A small electric screw driver works best for attaching the screws.

Secure the end screw on each half of the plate and then apply the remaining screws once the patch is in the desired location. See Figure 1

Three (3) screws on each side of the plate will usually hold the patch securely.

Once the patch is secured tighten the allen bolt with the foot standing on the ground bearing weight.

Should the patch be located on the inside quarter, adhesive hoof repair can be applied over the patch to make a smooth surface alleviating the possibility of stepping on the patch with the opposite foot.