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## Sole Protection & Support Material

Sole-Guard is the first product of its kind to offer fast effective protection and support for the unshod foot. Simply fill the bottom of the foot with Sole-Guard creating a protective coating that bonds to the foot up to three weeks. Sole-Guard sets firm to protect the unshod foot, yet retains flexibility to move naturally, providing comfort and support, in one easy application.

Sole-Guard is excellent for horses that are transitioning away from shoes, brood mares that need extra support late term and horses that just need extra protection while riding in rough terrain.

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Prepare foot as normal with proper trim and balance. Be sure to have the commissures of the frog clean and remove any sole that is ready to exfoliate.

Then DRY thoroughly. Use heat gun or hair dryer, even if you think that it's not necessary. Don't overheat.

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With contouring plastic held at the toe, hold the hoof up and dispense Sole-Guard onto the sole using enough material to cover the entire sole to the white line.

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After application, bring the plastic back from the toe to the heel smoothing and pushing material for even fill of sole and frog. Hold hoof for about 30 seconds.

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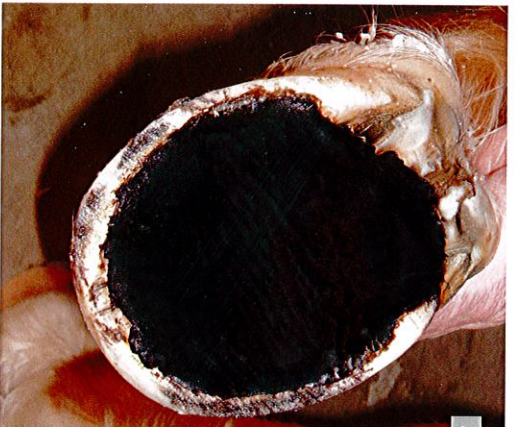
One minute after applying Sole-Guard, place foot down on smooth, hard surface. The hoof looks like this after weight bearing. Note even distribution of material.

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Hoof after final set, remove the plastic after 5 to 6 minutes, and this will be your result; support & protection!

Ref.	Beskrivning
46904	Konturfilm, rullar om 3 m.
Ref.	Description
46904	Contouring Plastic. Roll of 3 meters